SMARTY[®]
SAVER

SMARTY[®]
SAVERPLUS

SMARTY[®]
SAVERGEO

IT
EN
ES
Rev. AO/2021

WITHIN
EVERYONES'S
REACH

AED
DAE
DEA/DESA

Guida utente
User guide
Guía de usuario

 **AMI** ITALIA
MEDICAL DEVICES

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The device has been designed for use by lay personnel as well as by healthcare personnel who have duly completed and passed a BLSD course in accordance with international guidelines.

Use it if the patient:

- **Is unconscious and**
- **is not breathing and**
- **has no heartbeat**

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Being a life-saving device, the **AED** must always be available and ready to use; therefore the battery must always be connected to the device.

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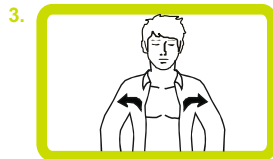
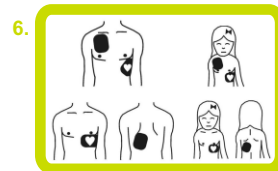
4. Press and Select
Adult or Pediatric

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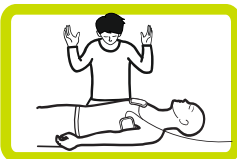
The Fully Automatic model will deliver the shock automatically, without pressing the shock button, after a warning voice message

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A metronome will scan the number and frequency of compressions

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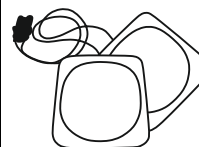
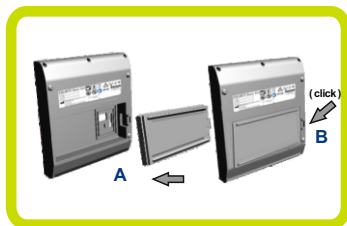
Start of the activation test, follow the voice messages provided by the AED

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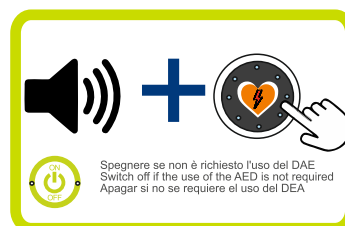
For the automatic model, pressing the shock button is not required, follow the AED's voice instructions

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The **Smarty Saver Plus** assists the operator in the correct execution of cardiac massage, during Cardio Pulmonary Resuscitation, thanks to the external CPR Quality sensor (SMT-C14034*). This external device is in fact able to measure the depth and frequency of the compressions performed and send this feedback to the Smarty Saver Plus device via Bluetooth.

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LED SCALE WITH PROGRESSIVE LIGHTING:

red LED lighting: insufficient or excessive pressure
red and orange LED on: pressure not yet sufficient
red, orange and yellow LED lighting: almost sufficient pressure
red, orange, yellow and green LED lighting: correct pressure

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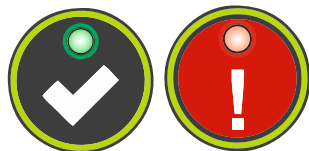
*see accessories section, CPR Quality Module SMT-C14034; for further information please refer to the specific user manual.

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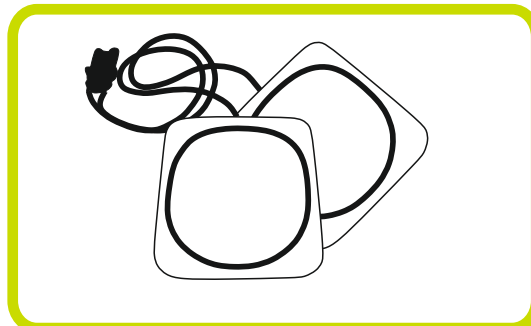
SMT-C2002

Elettrodi universali preconnessi face-to-face
Pre-connected universal electrodes face-to-face
Electrodos universales pre-conectados face-to-face

Prima di disconnettere la batteria accertarsi che il dispositivo sia spento da 5 secondi.
Before disconnecting the battery make sure the device is turned off for 5 seconds.
Antes de desconectar la batería, asegúrese de que el dispositivo se ha apagado durante 5 segundos.

Per informazioni dettagliate si faccia sempre riferimento al relativo manuale d'uso ed alle indicazioni riportate sulla confezione.
For detailed information, always refer to the relevant user manual and to the instructions on the packaging.
Para obtener información detallada, consulte siempre el manual de usuario correspondiente y las indicaciones del embalaje.

Modulo CPR Quality SMT-C14034 (per modelli Smarty Saver Plus/ Smarty Saver Geo)

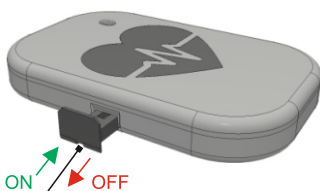
CPR Quality Module SMT-C14034 (for Smarty Saver Plus/ Smarty Saver Geo models)

Módulo de RCP de calidad SMT-C14034 (para los modelos Smarty Saver Plus/ Smarty Saver Geo)

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Flashing LED: Bluetooth signal search
Fixed LED: Bluetooth connection active

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External device capable of measuring the depth and frequency of the compressions performed and sending this feedback to the Smarty Saver Plus/Geo device via Bluetooth.

- Turn the module on by pushing the side ignition key
- Place it on the patient's chest at the time of CPR
- Perform compressions by checking that they are correct using the LED bar on the keyboard of the device and with the help of the AED voice instructions

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For all other information regarding the external Q-CPR sensor, please refer to the specific user manual.

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It is advisable to carry out routine maintenance on a daily basis; in the event of any anomalies, contact the service department.

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Thank you for choosing A.M.I. Italia **Smarty Saver Series** defibrillator!

For a correct use of the device, it is necessary, before using it, **to read carefully the relevant user manual (downloadable from www.amiitalia.com) and this user guide.** In order to ensure a correct and rapid traceability of the product and to receive information about all updates implemented, the user is required **to register the device** in the appropriate section of our website www.amiitalia.com.

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Gracias por elegir el desfibrilador **Smarty Saver Series** de A.M.I. Italia S.r.l.!